

## ►► How to use

1. Connect an Auto-Max to the handpiece and position the stopper on the Auto-Max.
2. The Auto-Max should meet the bone surface perpendicularly. Press the handpiece to fix the sharp point of the drill on the bone, and start drilling at about 300~500 rpm with copious irrigation.
3. Do not pump during harvest. Pumping may scatter the harvested bone.
4. The Auto-Max will automatically stop ped at bone depth of 4mm.
5. Disconnect the stopper from Auto-Max and collect particulated autogenous bone in a sterilized tray.

Repeat steps 1~5 until the desired volume of bone is obtained.

