


PDF Compressor Free Version

III. AnyRidge Prosthetic Kit

A kit with all kinds of driver that are needed for prosthetics.

Ref.C

KANPK3000

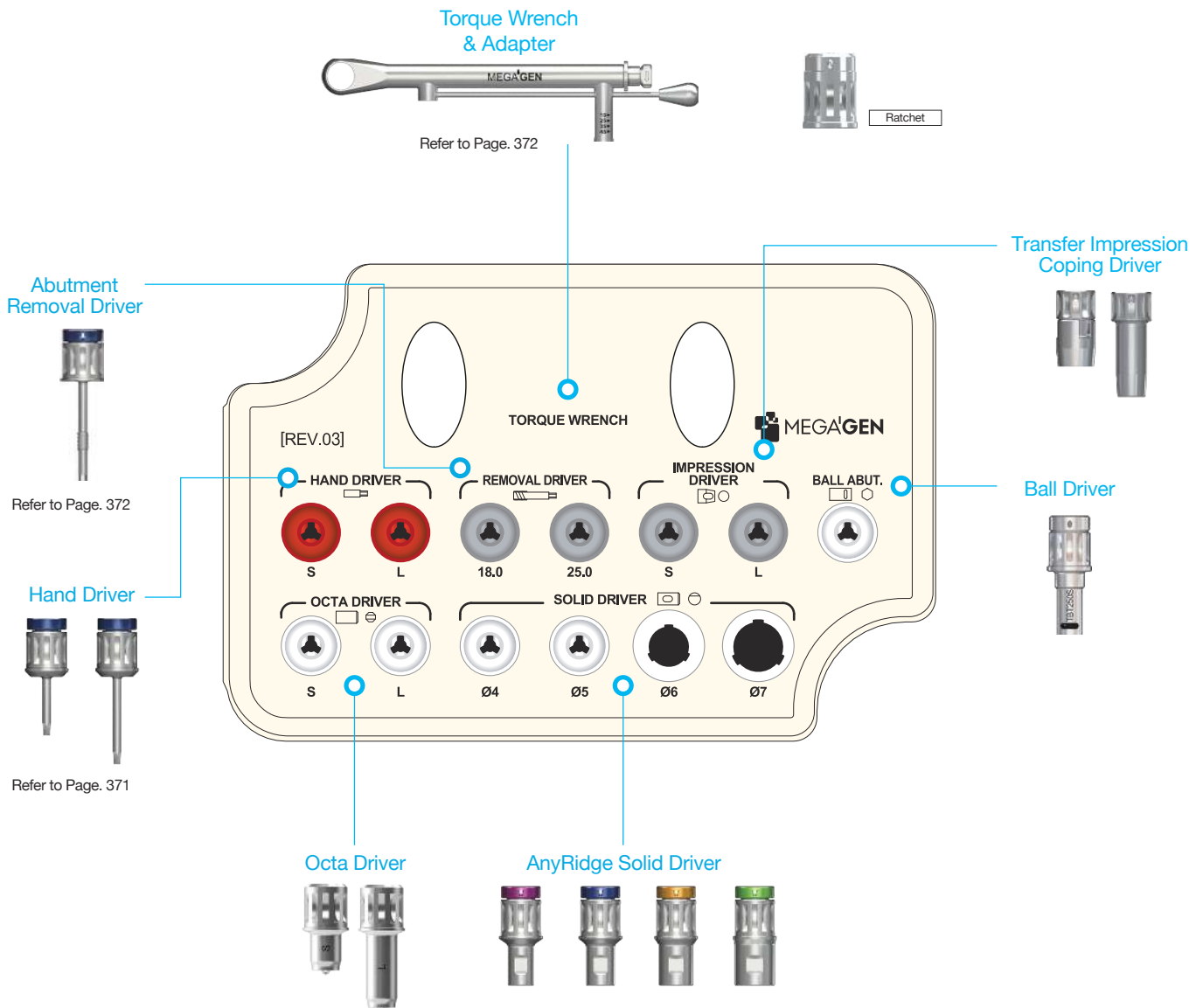

PDF Compressor Free Version

➔ Prosthetic Kit Components

Solid Driver

- For the delivery of Solid Abutments.
- Color coded for different profile diameters.
(Ø4-magenta, Ø5-blue, Ø6-yellow, Ø7-green)
- Two different heights. (8.5 / 13.5mm)
- Directly connectable to Torque Wrench.

Solid Abutment Profile Diameter	Length(mm)	Ref.C
Ø4	8.5	TANSDS400
	13.5	*TANS DL400
Ø5	8.5	TANSDS500
	13.5	*TANS DL500
Ø6	8.5	TANSDS600
	13.5	*TANS DL600
Ø7	8.5	TANSDS700
	13.5	*TANS DL700

(*) Separate sales item.



Octa Driver

- For seating of the Octa Abutment into the fixture.
- Can also be connected to Torque Wrench.

Length (mm)	Ref.C
7	MOD300S
13	MOD300L



Ball Driver

- For seating of the Ball Abutment into the fixture.
- Can connect to a Handpiece, Ratchet or Torque Wrench.
- Available in long and short.

Type	Ref.c
*Handpiece Connector(Short)	TBH250S
*Handpiece Connector(Long)	TBH250L
*Ratchet Connector(Short)	TBR250S
*Ratchet Connector(Long)	TBR250L
Toque Driver(Short)	TBT250S
*Toque Driver(Long)	TBT250L


(*) Separate sales item.



Impression Coping Driver (Transfer)

- For transfer type of Impression Coping.
- Works with friction only.
- Small but powerful grip.

Type	Ref.C
For Two piece impression Coping	TCMID
For One piece impression Coping	TCMIDE

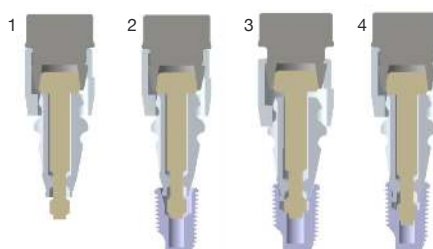


1. Connect Impression Coping and Impression Driver together
2. Adjust Connection with a Fixture by turning a Holder clockwise.
3. Push the Holder and put the Impression Coping into the Fixture.
4. Turn the Driver clockwise to ensure connection of the Impression Coping and Fixture.

PDF Compressor Free Version

IV. AnyOne Prosthetic Kit External / Onestage

Ref.C

MPK3000

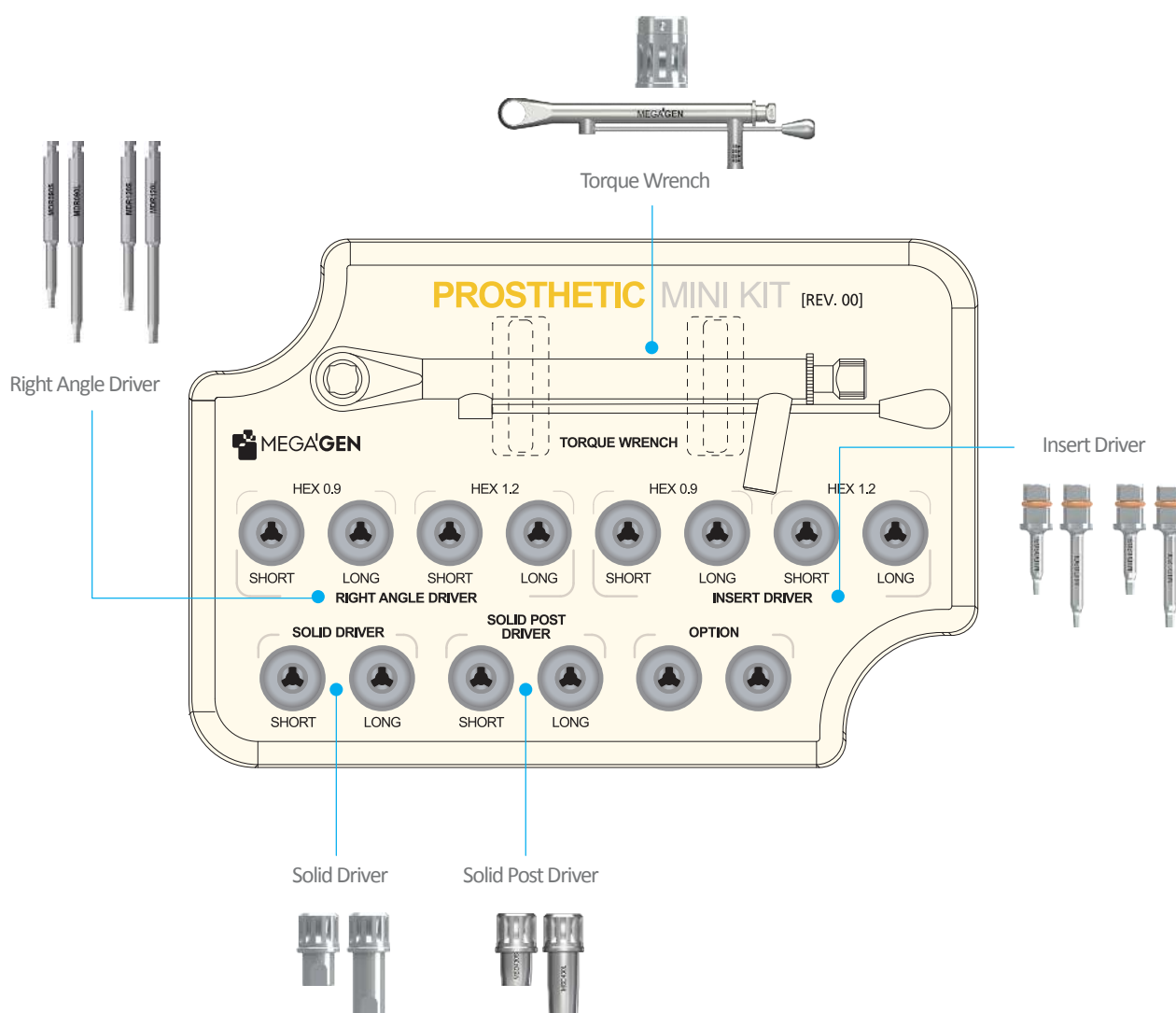


PDF Compressor Free Version

➔ Prosthetic kit Components

Right Angle Driver (hex 0.9)

- Can be engaged with Cover Screws, Abutment Screws and Healing Abutment
- Hex tip has been designed to stand Torque force of 35~45 Ncm
- Used for AnyOne External


Length(mm)	Type	Ref.C
4	*Ultra-short	MDR090SS
10	Short	MDR090S
15	Long	MDR090L

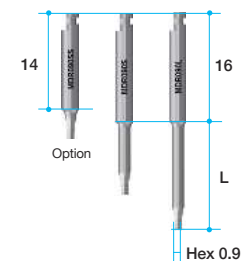

Right Angle Driver (hex 1.2)

- Can be engaged with Cover Screws, Abutment Screws and Healing Abutment
- Hex tip has been designed to stand Torque force of 35~45 Ncm
- Used for AnyOne Internal & External


Length(mm)	Type	Ref.C
4	*Ultra-short	MDR120SS
10	Short	MDR120S
15	Long	MDR120L
20	*Extra Long	MDR120EL

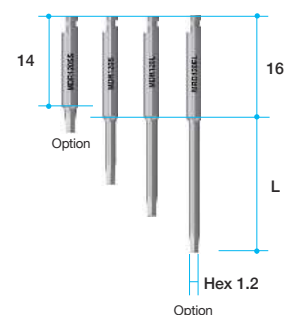

Insert Driver (hex 0.9)

- Can be engaged with Cover Screws, Abutment Screws and Healing Abutment
- Hex tip has been designed to stand Torque force of 35~45 Ncm


Length(mm)	Type	Ref.C
10	Short	MID090S
15	Long	MID090L

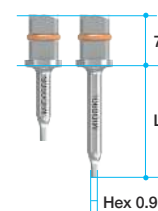

Insert Driver (hex 1.2)

- Can be engaged with Cover Screws, Abutment Screws and Healing Abutment
- Hex tip has been designed to stand Torque force of 35~45 Ncm

Length(mm)	Type	Ref.C
10	Short	MID120S
15	Long	MID120L

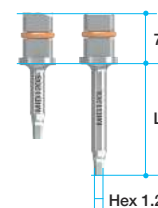

Solid Driver

- Can be engaged with Solid Abutment
- Connected to Torque Wrench as well
- Offers various options of Length (6mm / 12mm)

Diameter	Length(mm)	Type	Ref.C
Ø3,4	6	Short	MSD300S
	12	Long	MSD300L



Solid Post Driver

- Can be engaged with Solid Abutment
- Connected to Torque Wrench as well
- Offers various options of Length (6mm / 12mm)

Diameter	Length(mm)	Type	Ref.C
Ø4.3	6	Short	MSD430S
	12	Long	MSD430L

